Advanced Bionics

Advanced Bionics AG
Laubisrütistrasse 28
8712 Stäfa, Switzerland
+41 58 928 78 00

Manufactured by:
Advanced Bionics LLC
California, U.S.A.
+1 661 362 1400

AdvancedBionics.com

029-M649-85

029-M649-85 Rev A

© 2017 Advanced Bionics AG. All Rights Reserved.

SoundWave 3.1

CI-6055-011

instructions for use

English, Français, Español, Deutsch, Nederlands, Português

SoundWave 3.1 Professional Suite

Intended Use

SoundWave Professional Suite Software is used by a healthcare professional to program Advanced Bionics sound processors for use with Advanced Bionics cochlear implants.

Guidelines for IT Security

Because the SoundWave application runs on computers that can potentially be accessed by individuals not directly involved in the care of implant recipients, it is recommended that the following guidelines be followed for the computer(s) running SoundWave in order to help ensure the security and integrity of recipient data:

- Place in a physically secure location (e.g., locked to a work station)
- Ensure that all logins require a user-specific password
- Eliminate all guest accounts
- Enable automatic Windows updates
- Maintain an active physical network connection to ensure that automatic updates are deployed
- If Remote Desktop connections are allowed, only allow access with Network Level Authentication

In addition, it is recommended that the SoundWave database be deployed on a network, rather than on the local computer(s). The database server, whether on-site or in-cloud, should be in a physically secure location.

SoundWave™ Installation Instructions

SoundWave 3.1 includes new minimum software requirements. If you are running Windows 7, you will need to have Service Pack 1 installed. If you are running Windows Vista, you will need to install Service Pack 2. If you are running Windows 8, you will need to upgrade to Windows 8.1 Update 1. You can download these upgrades from Microsoft free of charge. In addition to meeting the requirements to run the operating system, the computer must have a minimum of 2GB hard disk space available if using a network database, and a minimum of 3GB hard disk space available if using a local database.

NOTE: *The service packs must be installed before you do a fresh install or upgrade to SoundWave 3.1. The application will not install without them.*

SoundWave 3.1 comes with the fitting manual stored electronically for easy access. You can find it in the Start Menu, listed under SoundWave 3.


This guide presents step-by-step instructions for installing the latest version of SoundWave.

NOTE: *If you plan to use or already have a network database on a remote server, it is recommended that you have your IT/database administrator run the database creation/backup/upgrade prior to installing/upgrading your programming computers.*


NOTE: The screenshots used to illustrate the following installation instructions may differ slightly from the pop-ups you see in your current version, but the installation process remains the same.

1. Backup Database

If you are upgrading from a previous version of SoundWave and you are using a local database, it is recommended that you backup your database before you upgrade. Note that SoundWave now requires your Windows User account to have Administrative rights in order to back up and restore a database. To back up the database, go to the Application dropdown menu and select Configuration ► Database Connections, as shown in the following image:

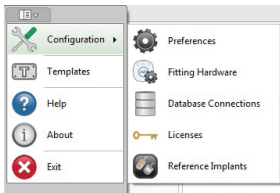
You will then see the following dialog box: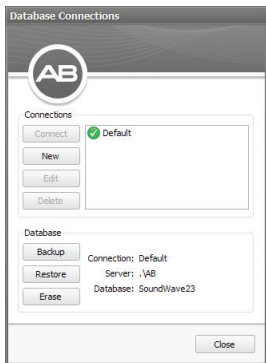

If you are currently using a remote network database, the 'Backup' option will be disabled. Otherwise, select Backup. After the backup completes, you will see the dialog box below.

NOTE: *In either case, please take note of the currently active database connection (indicated by a green check-mark). You may need this information after the upgrade has been performed.*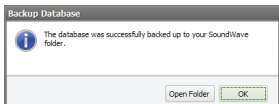

Your database is backed up. If you select Open Folder, you will see the directory in which it is stored.

2. Installation

To install or upgrade SoundWave 3.1, insert the SoundWave USB drive into your computer's USB port.

After a few moments, you may see the dialog box shown in Figure 1.

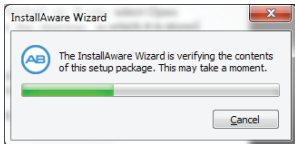
Figure 1

This should be followed by the Welcome dialog box (Figure 2). A language selector allows you to change the language of the installer itself. If the language choice is changed, it will be used on the following screen of the installer.


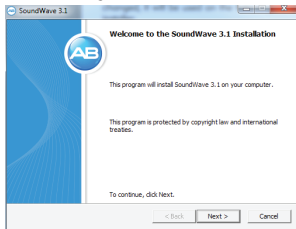
Figure 2

Click **Next** to continue.

If you are performing an upgrade from a previous version of SoundWave, you will be presented with the following confirmation dialog. Select **Yes** for the installation to continue.

NOTE: *This popup only appears in case of an upgrade*

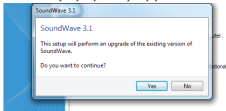3. License Agreement

This will take you to the End-User License Agreement dialog box (Figure 3).

Figure 3

Read the End-User License Agreement and check the box indicating you accept the terms of the agreement. Select the country in which you are located from the drop-down menu, then click on Next to continue.

NOTE: Depending on your computer's configuration, a restart might be necessary during the installation process. Once the computer restarts, the installation will either continue where it left off or you may need to restart it manually (executing SoundWaveSetup.EXE from the thumb drive).

You should then see the Installation Summary dialog box (Figure 4).

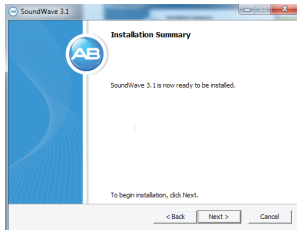
Figure 4

Click on Next to continue. At this point, the SoundWave 3.1 files will start copying onto the computer, and you should see the dialog box shown in Figure 5.

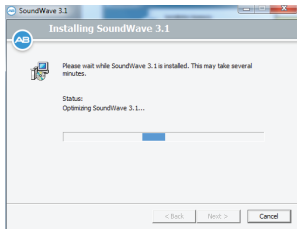
Figure 5

When the Next button is enabled, click on it to continue.

4. Finish Installation

After the installation of SoundWave 3.1 completes, you should see either the dialog box shown in Figure 6 or the one shown in Figure 7.

The SoundWave 3.1 installer will look to see if there is a local Advanced Bionics database instance already installed. If there is (typically because you were already using it with a previous version of SoundWave), you should see the dialog box in Figure 6. If so, just select Finish and the upgrade is complete. You should then go to Section 5, Upgrade Database.

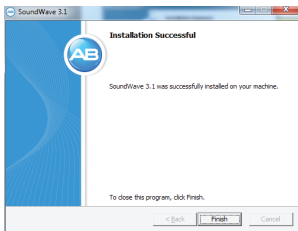
Figure 6

If there is not a local database instance, you will see the dialog box in Figure 7.

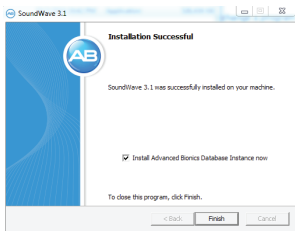
Figure 7

Independently of whether this is an upgrade or a new installation of SoundWave 3.1, you have the following two options:

- If you want to create a local database, select **Finish** with the **Install Advanced Bionics Database Instance now** checked. For details on the database instance installation, go to Section 6, New Database.
- If the plan is to connect to a remote network database, you will need to uncheck **Install Advanced Bionics Database Instance now** before clicking **Finish**.

At this point, SoundWave 3.1 has finished installing.

5. Upgrade Database

If you have upgraded from a previous version of SoundWave, you can now start running it. The first time you launch SoundWave 3.1, you should see the dialog box in Figure 8.

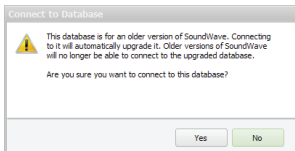
Figure 8

Selecting **Yes** will upgrade your existing database to the new version's format. Once complete, SoundWave 3.1 will be ready to use, with all your data migrated to the new format.

6. New Database

If you did not uncheck the **Install Advanced Bionics Database Instance now** in Figure 7, you should then see the dialog box shown in Figure 9, followed immediately by the Welcome box in Figure 10.

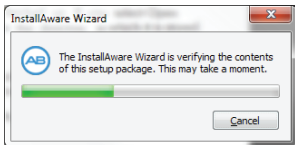
Figure 9

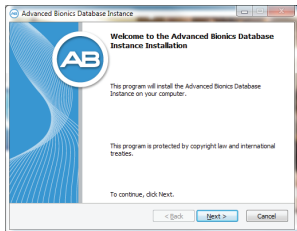
Figure 10

Click on **Next** to continue. You should then see the Installation Summary dialog box (Figure 11).

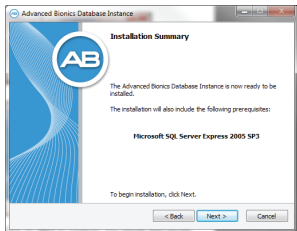
Figure 11

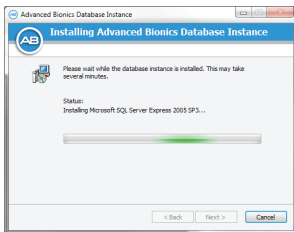
Figure 12

Click on **Next** to continue.

NOTE: Depending on your computer's configuration, a restart might be necessary during this process. Once it restarts, the installation will either continue where it left off or you might need to restart it manually (executing *ABDatabaseSetup.EXE* from the thumb drive).

You should then see the **Installation Successful** dialog box (Figure 13).

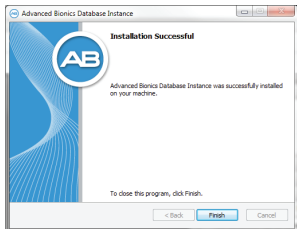
Figure 13

Click on **Finish** to complete the installation of the database instance.

The first time you start SoundWave 3.1, you will see the dialog box in Figure 14.

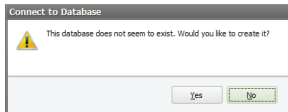
Figure 14

Click on **Yes** to continue. You should then see the SoundWave patient management window in Figure 15.

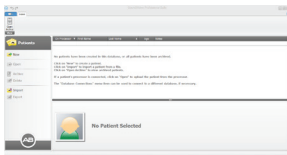
Figure 15

Your database has been created and you are ready to use SoundWave 3.1.

7. Troubleshooting Database Connection

If you have followed these steps and you do not see your database after launching SoundWave 3.1, you may need to reestablish a connection to it.

If you see the Database Connection message (Figure 16), click on Manage Database Connections. You will then see the Database Connections dialog box.

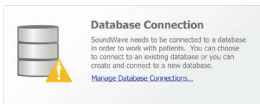
Figure 16

If you do not see the message in Figure 16, go to the Application dropdown menu and select Configuration ► Database Connections (Figure 17). This will launch the Database Connections dialog box (Figure 18).

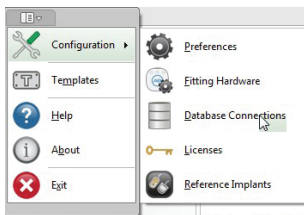
Figure 17

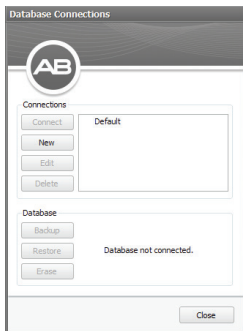
Figure 18

When you reach the dialog box in Figure 18, select the database connection that you were using in the previous version of SoundWave (this is the information you were asked to note during the preparation for the upgrade in Section 1, Backup Database, above).

Click **Connect** to complete the process.

CPI-3 Interface

The CPI-3 comes in a portable, compact package with fewer cables making it easier than ever to test, store and setup for programming. It comes with the ability to connect two processors to one interface for a quicker, more convenient and streamlined setup, especially for bilateral patients.

Connecting the CPI-3

Step 1: Start SoundWave software on the computer.

Step 2: Connect the CPI-3 to a USB port on the PC using the supplied USB cable.

Step 3: Connect the sound processor(s) to the CPI-3, using the programming cable(s) appropriate to the sound processor(s).

Step 4: Choose placement of CPI-3 for programming session: on the table, clipped onto the recipient's clothing or worn on a lanyard around the recipient's neck.

Warnings

In order to avoid delivering an overly loud stimulation to the patient:

- *Exercise caution when making large increases in M levels, especially while stimulating the patient.*
- *Exercise caution when making large increases in M levels in interpolation mode, especially while stimulating the patient.*

Patients should be advised to use the programmed C1 Harmony sound processor only with the implant that was stimulated during fitting.

You should instruct those patients capable of understanding your communications to remove the headpiece if uncomfortable stimulation is encountered.

Technical Support

Note that programming may help alleviate the side effects associated with open electrodes, short electrodes, and collateral stimulation.

Advanced Bionics has a toll free technical hotline phone number (877-271-6727) where AB audiologists may be contacted from 5:00AM to 5:00PM Monday through Friday Pacific Time to answer questions about SoundWave, related hardware, or other technical issues.

If you are outside the U.S. and Canada, technical support is provided locally. If you have any questions about SoundWave, related hardware or other technical issues, please contact your local Advanced Bionics representative.

Compatibility

Equipment Minimum Requirements	
Operating System	Windows 10 • Windows 8.1 Update 1 • Windows 8.1 Pro Update 1 • Windows 8.1 Enterprise Update 1 • Windows VISTA SP2 • Windows 7 SP1
Memory	1 GB
Hard Drive	2 GB, (Plus 1 GB for database if local)
Graphical Resolution	1280 x 720 24-bit color
Communication Port	USB 1.1, 2.0, 3.0 + COM port for CPI 2
Fitting Platform Specifications	
.NET Framework	4.6.1 or higher
Database Server	SQL through Microsoft SQL Express/SQL Server 2005, 2008, 2012

SoundWave 3.1	
Implant Type	CI ¹ • CII • HiRes™ 90K Family • HiRes™ Ultra
Processor Type	PSP • Harmony • Neptune • Naída CI Q30 • Naída CI Q70 • Naída CI Q90

¹Only on Harmony

Advanced Bionics AG
Laubisrütistrasse 28, 8712 Stäfa, Switzerland
T: +41.58.928.78.00
F: +41.58.928.78.90
info.switzerland@AdvancedBionics.com

Advanced Bionics LLC
28515 Westinghouse Place Valencia, CA 91355,
United States T: +1.877.829.0026
T: +1.661.362.1400
F: +1.661.362.1500
info.us@AdvancedBionics.com

For information on additional AB locations, please visit
AdvancedBionics.com/contact

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Para mais informações sobre outros endereços da AB,