

1. This document was created to support maximum accessibility for all learners. If you would like to print a hard copy of this document, please follow the general instructions below to print multiple slides on a single page or in black and white.
2. This handout is for reference only. Non-essential images have been removed for your convenience. Any links included in the handout are current at the time of the live webinar, but are subject to change and may not be current at a later date.
3. Copyright: Images used in this course are used in compliance with copyright laws and where required, permission has been secured to use the images in this course. All use of these images outside of this course may be in violation of copyright laws and is strictly prohibited.
4. Social Workers: For additional information regarding standards and indicators for cultural competence, please review the NASW resource: [Standards and Indicators for Cultural Competence in Social Work Practice](#)
5. Need Help? Select the “Help” option in the member dashboard to access FAQs or contact us.

How to Print Handouts

- On a Mac
 - Open PDF in Preview
 - Click File
 - Click Print
 - Click dropdown menu on the right “preview”
 - Click layout
 - Choose # of pages per sheet from dropdown menu
 - Checkmark Black & White if wanted.
- On a PC
 - Open PDF
 - Click Print
 - Choose # of pages per sheet from dropdown menu
 - Choose Black and White from “Color” dropdown

No part of the materials available through the continued.com site may be copied, photocopied, reproduced, translated or reduced to any electronic medium or machine-readable form, in whole or in part, without prior written consent of continued.com, LLC. Any other reproduction in any form without such written permission is prohibited. All materials contained on this site are protected by United States copyright law and may not be reproduced, distributed, transmitted, displayed, published or broadcast without the prior written permission of continued.com, LLC. Users must not access or use for any commercial purposes any part of the site or any services or materials available through the site.



Beltone TV-Streamer+



61

New Pairing Sequences

Step # 1



Pair hearing aids to TV-Streamer+

Step # 2



Pair hearing aids to phone and Beltone HearMax app

Step #3



Pair TV-Streamer+ to Beltone HearMax app

Beltone UNIVERSITY

62


LED Light Pattern

Function	Pattern	Pattern description
Streaming		Constant ON
Not streaming (standby)		Constant OFF
Pairing mode active		Repeated flash (ON 200 ms, OFF 500 ms)
Successful pairing		Constant ON for 3 seconds
Firmware update ongoing		Repeated flash (ON 100 ms, OFF 100 ms)
Firmware update complete		Constant ON for 3 seconds
Adjustable delay mode active		Repeated flash (ON 1500 ms, OFF 1500 ms)
Error (service needed)		3 flash sequences (ON 200 ms, OFF 200 ms, ON 200 ms, OFF 200 ms, ON 200 ms, OFF 1000 ms)



63

Pairing in Beltone Solus Max

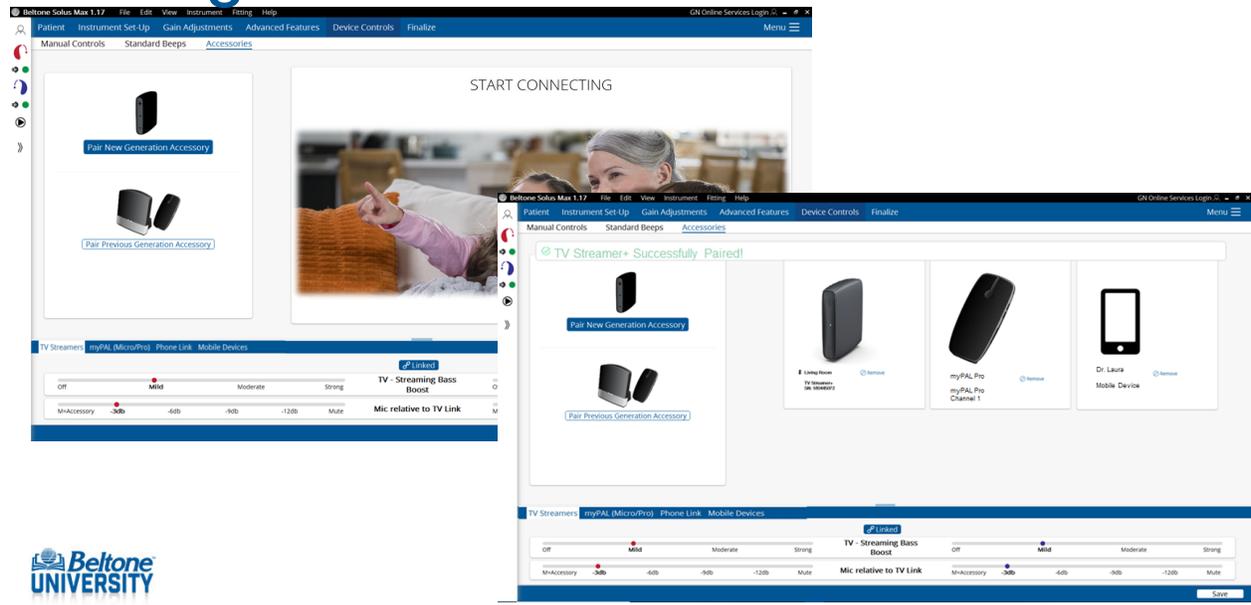


64

Accessory Mix-in Streaming

Beltone Serene

All-Around	Hear in Noise										
All	250	500	750	1K	1.5K	2K	3K	4K	6K	8K	
50	0	0	0	2	10	14	16	11	8	4	
70	0	0	0	0	6	10	11	7	5	2	
90	0	0	0	0	0	0	0	0	0	0	
CR	1	1.1	1.3	1.5	1.7	1.6	1.6	1.5	1.2	1.1	
MPO	96	99	102	104	108	111	112	112	110	110	
Handles:	6		10								

Beltone Achieve and older

All-Around	Ultra Focus 2	Streaming Accessories									
All	250	500	750	1K	1.5K	2K	3K	4K	6K	8K	
50	0	0	0	2	9	11	14	9	9	7	
70	0	0	0	0	4	7	9	5	6	4	
90	0	0	0	0	0	0	0	0	0	0	
CR	1	1.1	1.3	1.5	1.5	1.4	1.5	1.3	1.4	1.2	
MPO	96	99	102	104	107	109	110	109	108	110	
Handles:	6		10								




65

Number of Paired Devices

Electronic devices

Bluetooth LE audio, MFi and ASHA

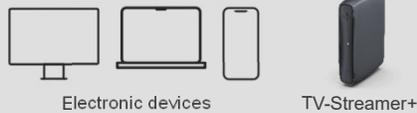

Up to 8 devices

TV Link 2/myPAL

Previous generation of accessories



Up to 3 in total

Phone Link 2

Connects to phones with Classic Bluetooth



1 unit

Remote Control




1 unit

66

Pairing Hearing Aids to Current Phones

iPhone

Android (Pixel)

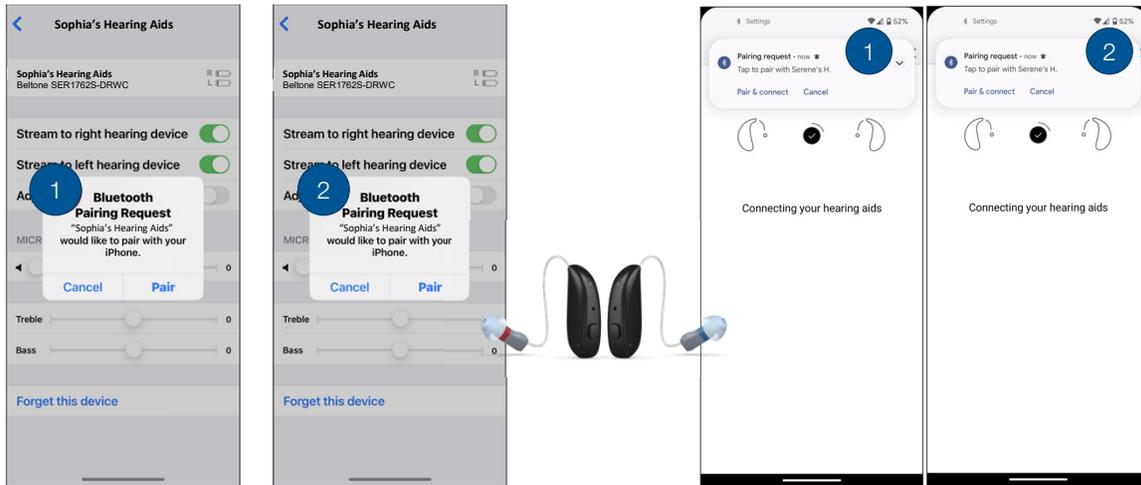


67

Pairing Hearing Aids to Bluetooth Low Energy Audio devices


iPhone

Android (Pixel)

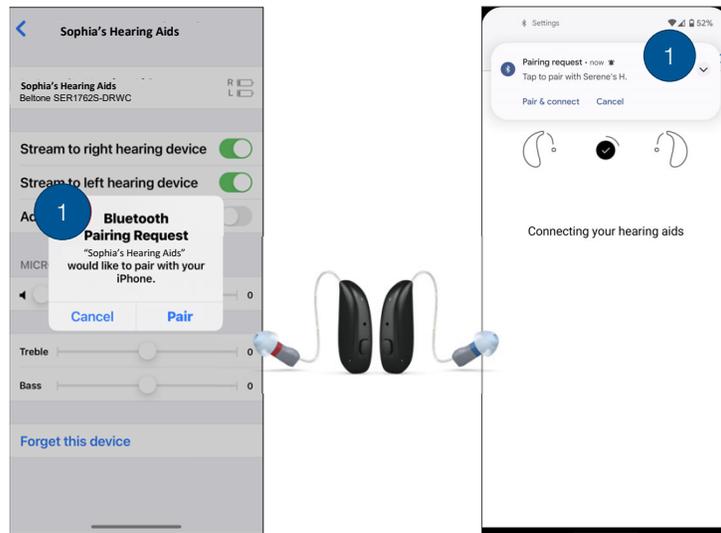


68

Pairing TV-Streamer+ to **Beltone HearMax** app

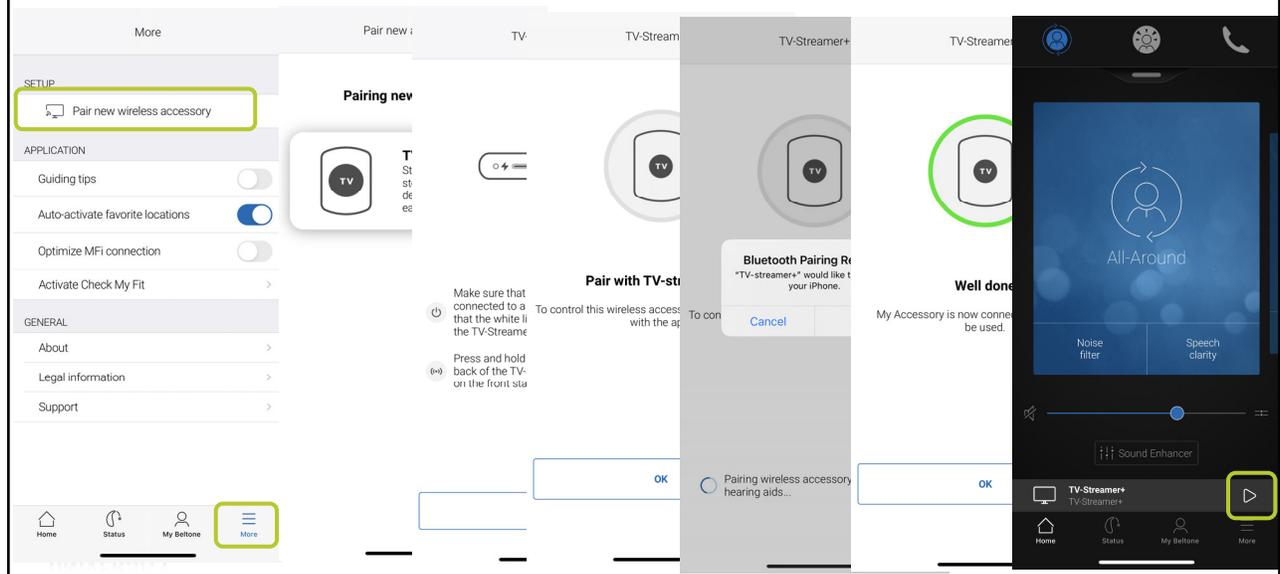


69

How to Start Streaming

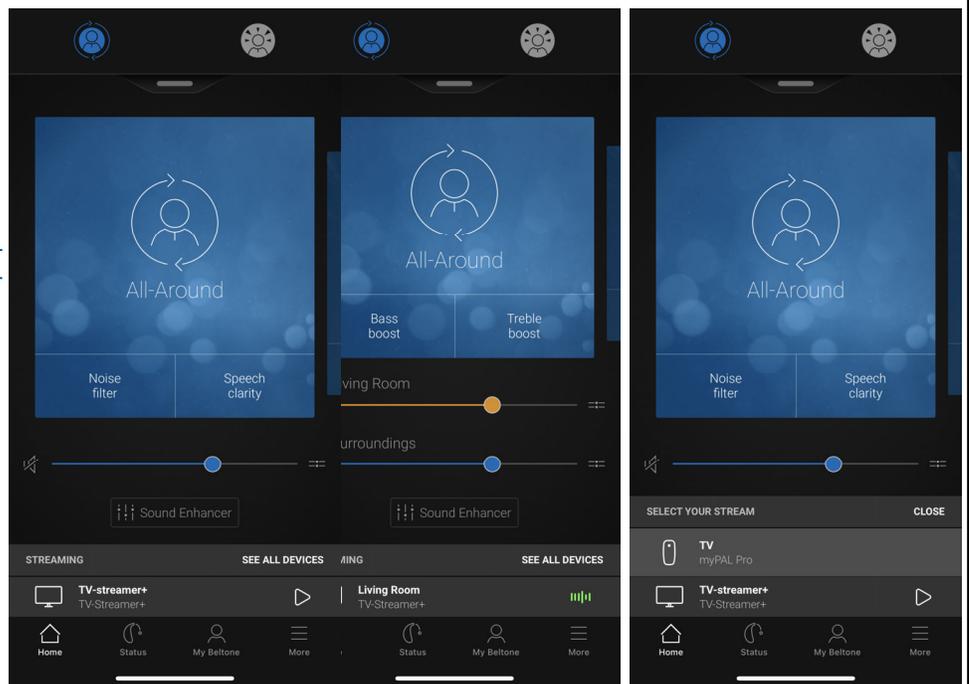


70

Renaming the TV-Streamer+

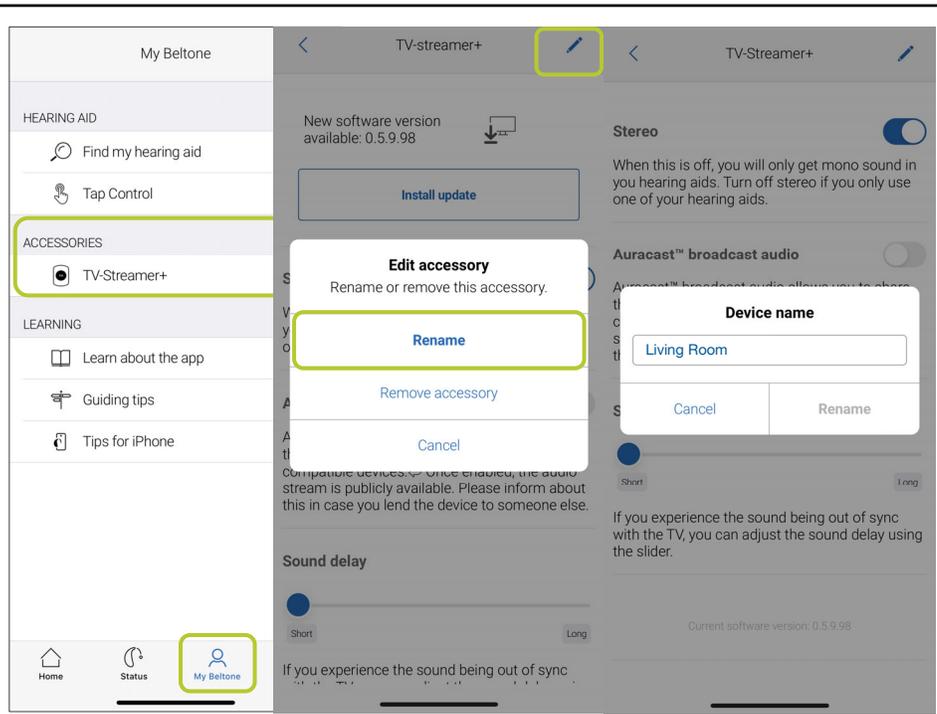

71

TV-Streamer+ Settings

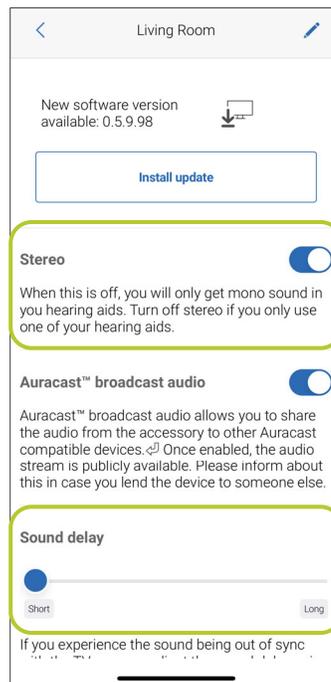

72

Firmware Update

My Beltone < Living Room Software update Software update Software update

HEARING AID

- Find my hearing aid
- Tap Control

ACCESSORIES

- Living Room

LEARNING

- Learn about the app
- Guiding tips
- Tips for iPhone
- Check My Fit

New software version available: 0.5.9.98

Install update

Installation takes a few minutes

Stay in the Beltone HearMax app while in

Please be aware of the following:

- Do not uninstall this app during software installation.
- On rare occasions, the installation may fail if the current software cannot be restored. In this case, you will be asked to return to your hearing aid to restore your hearing aid.

Installation successful

You can now continue to use your accessory with the new software.

Sound delay

Short

If you experience the sound being out of sync...

Continue **Accept and Continue** **Close**

73

Accessory Updater

Beltone Accessory Updater

Accessory Updater

Searching for Bluetooth Accessories

TV-Streamer+ Serial: 2381046581 **Update**

Accessory Updater

BLE

TV-Streamer+ **Check BLE Updates**

USB

Check USB Updates

Accessories supported:

- myPAL
- myPAL Micro
- myPAL Pro
- Audio Streamer
- Phone Link
- Remote Control 2

Accessory Updater

Accessory: TV-Streamer+

Serial: 2381046581

Version: 0.5.10.14

Firmware Updated


Find

Back

Beltone UNIVERSITY

74

Auracast in the Beltone HearMax app

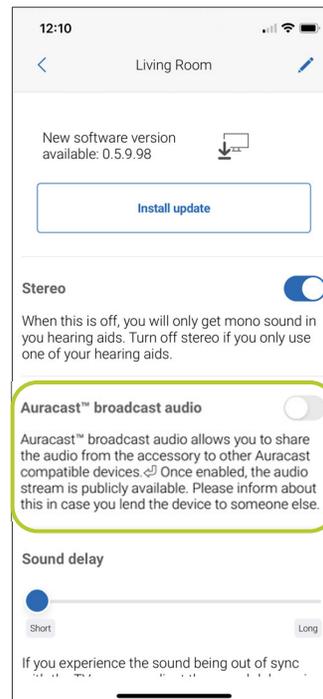

75

Learning Summary



Our new TV-Streamer+ has a smaller sleeker design.



Pairing is easier with MagicPairing, and when paired to the Beltone HearMax app patients have addition controls.



From their app patients can adjust the stereo sound, A/V delay and run a firmware update. Patients can also set their TV-Streamer+ to be an Auracast transmitter.



76