

CROS/BiCROS Fitting Push Button Configuration

Side Configured	Change Program	Volume Up/ Volume Down	CROS Up/ CROS Down	Activation of Streaming	Minimum Volume	Mute/Unmute
CROS Transmitter	Changes program on hearing aid side	Not applicable to CROS Transmitter	Increase/decrease CROS Transmitter sound balance	Activates accessory streaming to hearing aid side	Sets minimum CROS sound balance setting	Mutes/Unmutes CROS Transmitter
Hearing aid side	Changes program on hearing aid side	Increase/decrease hearing aid volume	Not applicable to hearing aid side	Activates accessory streaming to hearing aid side	Sets minimum hearing aid volume	Mutes/Unmutes hearing aid side
Both CROS Transmitter and hearing aid side	Changes program on hearing aid side	Not applicable – does not apply to CROS Transmitter	Not applicable – does not apply to hearing aid side	Activates accessory streaming to hearing aid side	<ul style="list-style-type: none"> • Sets minimum CROS sound balance setting if CROS push button is used • Sets minimum hearing aid volume if hearing aid side push button is used 	<ul style="list-style-type: none"> • Mutes/Unmutes CROS Transmitter if CROS Transmitter push button is used • Mutes/Unmutes hearing aid if hearing aid side push button is used