

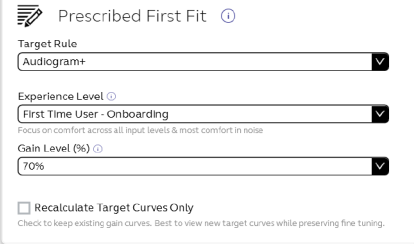

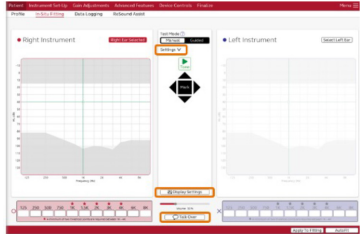
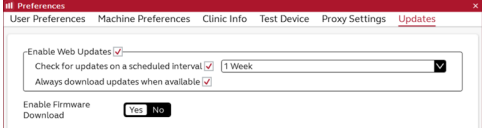
# ReSound Smart Fit 1.17

We continuously strive to improve the fitting experience for you and your patients. Based on your feedback, ReSound Smart Fit™ now supports more accurate fittings and a faster workflow.

## What's new?

- Support for new ReSound products, including:
  - ReSound Nexia family
  - CROS/BiCROS solution
  - TV Streamer+ accessory
- TapControl
- “Hear In Noise” program
- User Profile, “First Time User - Onboarding”
- Improved AutoREM experience
- Improved In-Situ Fitting experience
- Over the Air (OTA) minor firmware update feature

What's new	Feature	Benefit	Available For	Image
<b>CROS/BiCROS Solution</b>	Connecting ReSound Nexia CROS Transmitter to control the balance of CROS Transmitter relative to hearing aid/receiver side and more.	CROS/BiCROS solution for patients with single-sided deafness	All ReSound Nexia hearing aids	
<b>TV-Streamer+ Accessory</b>	Pairing next generation wireless accessory, TV-Streamer+, which is smaller, lighter, and easier to mount and connect. Bluetooth® Auracast™ ready.	Patients enjoy clear streamed sound from TV or other audio source. Bluetooth Auracast allows broadcasting to multiple compatible devices.	All ReSound Nexia hearing aids	
<b>TapControl</b>	TapControl enables the patient to answer calls by double-tapping the top of the hearing aid or their ear to stream the call directly to their hearing aids.	A discreet and convenient way to answer a call wirelessly when the devices are paired with a compatible smart devices*.  *For a complete list of compatible devices, visit <a href="https://resound.com/compatibility">resound.com/compatibility</a> .	ReSound Nexia Micro RIE (60S) hearing aids	
<b>'Hear in Noise' Program</b>	Hear in Noise program replacing Front Focus, Ultra Focus and Restaurant programs.	Consistent and simple offering of listening programs, with one superior program specifically intended for hearing in noise.	All ReSound hearing aids in ReSound Smart Fit	

What's new	Feature	Benefit	Available For	Image
<b>First Time User – Onboarding' Patient Profile</b>	User profile named 'First Time User – Onboarding' replaces the existing 'Comfort User' profile.	Give your patients a smooth start to hearing – and an alternative if the 'First Time User' profile is perceived as too aggressive by your patient.  Includes UI updates to better guide you to which profile might suit your patient best for a great first fit.	All ReSound hearing aids in ReSound Smart Fit	
<b>Improved AutoREM</b>	New fine-tuning options to support existing functionality: <ul style="list-style-type: none"> <li>• <b>Match Target</b> option: Finetune gain to target for 3 inputs</li> <li>• <b>Quick Run</b> option: Existing option to run the soft and loud input for recording and observation purposes</li> <li>• <b>Mix and Match</b>: Choose to Target Match or Quick Run for soft and loud inputs based on your needs and patient tolerance</li> </ul>	Easily find AutoREM and the finetuning options with fewer clicks.  Allows you more control over how you want to run AutoREM – by either using existing functionality (Quick Run), get more accurate low and high intensity results (Match Target) or even mix and match what type of curves to run.	All ReSound hearing aids in ReSound Smart Fit	
<b>Improved In-Situ Fitting</b>	Improved process flow with fewer clicks to get through testing	More seamless and easy navigation with no compromise on functionality.	All ReSound hearing aids in ReSound Smart Fit	
<b>Over the Air (OTA)</b>	Enables Smart Fit fitting software to download the latest version of hearing aid firmware to the device.	Allow updating patients' hearing aids with the latest firmware version without updating the version of Smart Fit fitting software.	All ReSound hearing aids in ReSound Smart Fit	

If you have any questions about ReSound Smart Fit 1.17 and the new updates, please contact your ReSound sales representative.

**Manufacturer according to FDA:**  
**GN ReSound North America**  
8001 E Bloomington Freeway  
Bloomington, MN 55420  
USA  
1-800-248-4327  
pro.resound.com

**Manufacturer according to Health Canada:**  
**ReSound Canada**  
2 East Beaver Creek Road, Building 3  
Richmond Hill, ON L4B 2N3  
Canada  
1-888-737-6863  
pro.resound.com

